**Instructions for Use** English

***Warning:*** This tool is indicated for embryo biopsy which may be done in order to perform preimplantation genetic diagnosis (PGD) on the genetic material in the biopsied cell(s). Tests for PGD are currently developed and their performance characteristics are determined by individual laboratories for their own use. Currently, these tests have not been cleared or approved by the Food and Drug Administration.

***Warning:*** These tools are intended for one-time use only and must remain covered until ready to use in order to maintain both sterility and to prevent contamination. The user is warned to follow safe, established laboratory practices in order to prevent accidental DNA contamination after the micropipette has been removed from its protective covering.

**Indications for Use:** The Blastomere Biopsy Pipettes are tools used in IVF/Assisted Reproduction Technology laboratories for removal of blastomere(s) from embryos, which may be done in order to perform preimplantation genetic diagnosis on the genetic material in the biopsied cell(s).

The Polar Body Biopsy Pipettes are tools used in IVF/Assisted Reproduction Technology (ART) laboratories for removal of polar bodies from oocytes, which may be done in order to perform preimplantation genetic diagnosis on the genetic material in the biopsied cell(s).

Notes:

2-cell MEA tested: Acceptable at ≥ 80% blastocysts per embryo. Endotoxin tested: LAL acceptable at ≤ 0,5 endotoxin units per device.

**Instructions for Use:**

1. Micropipettes should be used by qualified personnel.

2. Micropipettes are for single use only. The used pipettes are contaminated and considered as biohazard materials. They should be disposed properly.

3. Remove a micropipette from its packaging just before use.

4. Check the package for damages before use. Sterility is no longer guaranteed if the pouch is damaged or opened.

5. The pouch can be opened by tearing off one end or by cutting using a pair of scissors.

6. To open the case, hold the case as shown in Figure 1. Make sure the logo engraving is up facing your right hand.

7. Using your left index finger and thumb press the case on both sides marked “lIII” then using your right index finger and thumb pull the top cover as shown in Figure 2.

8. Hold firmly the pipette in the middle and gently pull it out from the base as shown in Figure 3

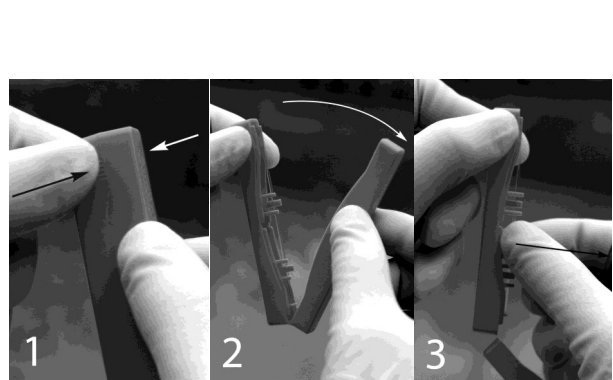
Figure 1

Figure 2

Figure 3

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